

# BIAKŌS™

ANTIMICROBIAL  
WOUND GEL



## Instructions For Use

Unscrew cap, remove foil seal, then screw cap back on. Open flip cap and squeeze gently to apply a generous amount of gel directly to the wound. Close flip cap after use. Secondary dressing may be used. Change dressings every 24-72 hours or as directed by a clinician.

## Storage

Store with flip cap closed at room temperature. Avoid excessive heat. DO NOT refrigerate or freeze. Discard 56 days after opening.

## Ingredients

Water, Poloxamer 407, Sodium Chloride, Ethylhexylglycerin, Octane-1-2-diol, Polyaminopropyl Biguanide (preservative: PHMB: 0.1% w/w), Edetate Disodium (EDTA), Edetate Trisodium (EDTA)

## Caution

Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.

## Glossary



Do not use if package is damaged



Single-patient use



Consult instructions for use



Store at room temperature

## Description

BIAKŌS Antimicrobial Wound Gel provides a moist environment to wound surfaces. BIAKŌS Antimicrobial Wound Gel is a safe and gentle colorless gel. The gel provides preservative properties through the antimicrobial PHMB.

BIAKŌS Antimicrobial Wound Gel:

- Resists microbial colonization within the gel during shelf storage.
- Provides an amorphous gel covering.
- Facilitates autolytic debridement through a moist wound environment. Wounds experience some level of autolytic debridement in which the body's own enzymes breakdown necrotic tissue.

## Indications

BIAKŌS Antimicrobial Wound Gel is intended for the management of ulcers (including diabetic foot and leg ulcers and pressure ulcers), first- and second-degree burns, partial- and full-thickness wounds, large surface area wounds, and surgical incisions.

## Warnings

For EXTERNAL USE ONLY on skin and wounds. DO NOT use for infusion or injection. DO NOT use in eyes. DO NOT ingest. DO NOT contaminate opening. DO NOT use if you have an allergy to one or more of the ingredients. DO NOT use on exposed cartilage. DO NOT use or mix with other cleansers, soaps, lotions, or ointments. DO NOT use in the middle or inner ear. NOT evaluated for use in the pediatric population. Accumulated use time should not exceed 30 days.

### Distributed by:

Wound Care Innovations, LLC,  
a Sanara MedTech Inc. Company  
1200 Summit Ave., Suite 414  
Fort Worth, TX 76102  
1-800-205-7719

BIAKŌS is a trademark of  
Wound Care Innovations, LLC,  
a Sanara MedTech Inc. Company.  
© 2020. All rights reserved.  
Patent on file. Rev. 200316

[Learn more at biakos.net](http://biakos.net)